

Tango XXL shower toilet chair

Instruction card

5100 5850
Tango XXL

01. Safety recommendations

- Read instruction for use before use (www.lopital.com).
- Maximum load 360 kg / 793.7 lbs.
- Make sure that the chair is on the brake during client transfers from and to the device.
- Place both feet on the foot supports.




02. Use

1. Place the chair in the right position and turn footrests away.
2. Apply the brakes to the chair!
3. If necessary, fold the armrests up.
4. Support the person with disability when they are taking a seat.
5. Shut the armrests and let the client's feet rest on the footplates.

Stand up by carrying out the same procedure in reverse order.

03. Removable seat

If required, you can remove the Tango XXL seat from the frame. This allows the seat to be cleaned thoroughly, as no dirt can accumulate between the frame and the seat.

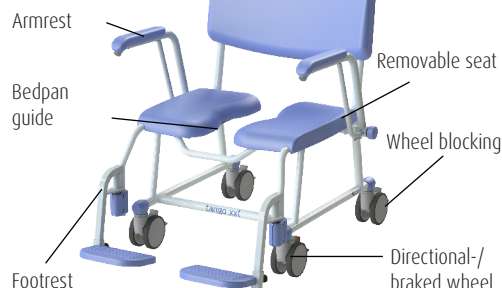


06. Footrest

Both foot plates must be turned away for the person with disability to take a seat in the chair. Make sure that the person places both feet on the foot plates when the chair is moving. The footrests are adjustable to 3 width positions. Lift the footrest and turn it outwards. The footrest automatically locks into one of the three positions using a locking pin.




04. Wheel blocking

1. Wheel blocking.
2. Unlocking.

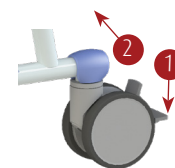

05. Directional- / braked wheel

1. Wheel blocking
2. Unlocking

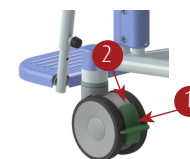

07. Armrest

1. Unlocking
 2. Press the blue button
 3. Turn backwards
- To close, follow the above steps in reverse order.



The armrests of the Tango XXL are width-adjustable. Loosen the wing knob on the armrest and slide the armrest to the desired position. Once the correct position has been reached, securely tighten the wing knob.

08. Toilet function

- *In combination with toilet:* the Tango XXL can be moved over almost every toilet.
- *With bedpan:* the Tango XXL can also be used as a toilet chair. Simply slide the stainless steel / plastic bedpan under the chair using the guides.

09. Cleaning and maintenance

The Tango XXL must be cleaned after every use, with water and regular household cleaning agents. Do not use any abrasive or other aggressive cleaning products. If necessary, the PUR components can be disinfected with a solution of 70% ethanol or propanol. The complete maintenance schedule and cleaning protocol can be found in the instructions for use (www.lopital.com).

According to the regulation MDR (EU) 2017/745 for medical devices, a chair must be maintained in line with the manufacturer's instructions. Those instructions can be found in the maintenance protocol, see website Lopital (www.lopital.com).

Important!

This quick instruction card may not be considered a replacement for the instruction for use!

In the event of a breakdown, contact Lopital.
Tel: +31 (0) 13-5239300
E-mail: info@lopital.nl

Instruction for use
For the full instruction for use go to: www.lopital.com

