

P:\SW-Ftek\Timo en Cleo\Timo 6036

Signed	KA	Date	02-03-2016
Revised	- /KA	Date	-
Revised	- /KA	Date	-

Overview parts Timo


Copyright reserved by law Unit of measurement: mm

Lopital Nederland bv

Type nr: 54006036

Timo 54006036

Part nr.	QTY.	Art.nr.	Part
1	1	70203501	Frame
2	1	70203503	Seat
3	1	70203509	Backrest
4	4	70519928	Castor Ø 125 mm braked
5	1	70603602	Hinge seat
6	1	70565001	Armrest
7	1	69608024	Footrest
8	1	69501104	Bed pan + lid
9	4		Cap castor
10	4		Tube cap
11	4		Capring M4
12	4		Chipboard screw M4x30
13	4		Screw cap M4
14	2		Cylinder head screw M8x12
15	4		Washer M6
16	4		Chipboard screw M5x25

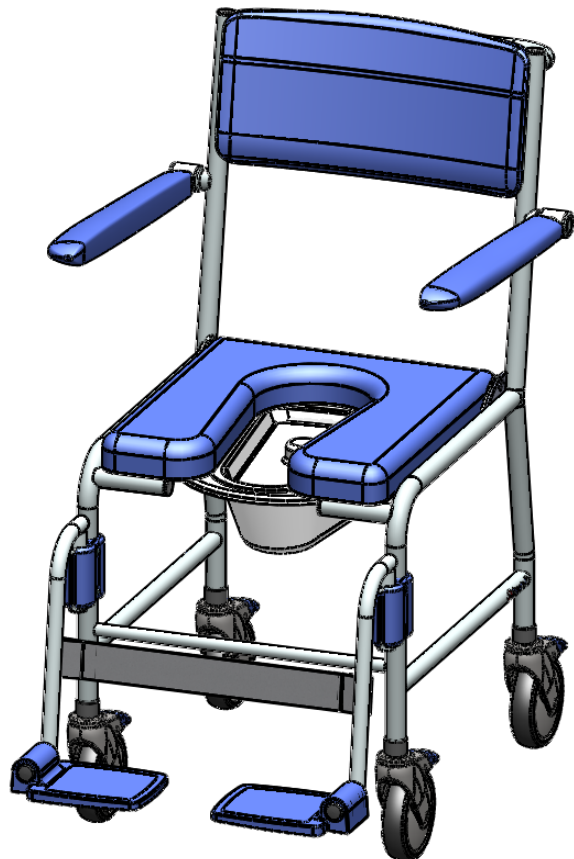

P:\SW-Ftek\Timo en Cleo\Timo 6036\

Signed	KA	Date	02-03-2016
Revised	- /KA	Date	-
Revised	- /KA	Date	-

Overview parts Timo

Copyright reserved
by law

Unit of measure-
ment: mm

Lopital Nederland bv

Type nr: 54006036