

P:\SW-Ftek\Tango XXL 5850\

Drawn by: KA	Date: 26-11-2025
Revised: 1 /KA	Date: 20-02-2026
Revised: - /KA	Date: -

Overview parts Tango XXL

Copyright reserved by law Unit of measurement: mm

Lopital Nederland bv

Draw. no.: EXPV5850-2025-01

Tango XXL, Article no: 5100 5850

Part no.	Art.no.	QTY.	Name	Drawing no.
1	70585001	1	Seat frame	5850-11
2	68405900	1	Armrest set width adjustment XL/XXL	Armlegger breedteverstelling XL XXL
3		1	Footrest set width adjustment XL/XXL	Voetsteunset XXL
4	70590009	2	Seat part XXL	5900-30
5	70590010	1	Backrest XXL	5900-31
6	70560066	4	Protective cover castor	5600-48
7	70519931	2	Clamp set footrest blue	5500-24
8	70590015	1	Guide for po set XXL	5900-25
9	70590021	3	Castor double roll Ø125 braked	Blickle Zwenkwiel XXL
10	70590022	1	Direction castor double roll Ø125 braked	Blickle Zwenkwiel XXL
11	70602006	8	Snaploc	Snaploc 4029 007 0601
12	70820019	4	Cross Knob PA 40 mm M6 grey	Kruisknop M6
13	70590049	2	Wing screw M8	Kipp K0274.0108X20
14		2	Tube cap Ø30	Insteekdop rond 32 grijs
15		4	Screw M5x6 ISO7380 A2	M5x6 ISO7380
16		8	Screw M6x10 DIN965 A2	M6x10 DIN965
17		4	Screw M8x35 DIN912 A2	M8x35 DIN912
18		4	Screw M10x70 DIN912 A2 Dry-loc + Lurinox	M0x70 DIN912 Dry-loc+Lurinox
19		4	Washer M5 Ø15 DIN9012 A2	Carrosseriering M5 DIN9021 A2
20		4	Nord-Lock M10	Nord-Lock M10
21		4	Locknut M8 DIN985 A2	M8 DIN985



Drawn by: KA	Date: 26-11-2025
Revised: 1 /KA	Date: 20-02-2026
Revised: - /KA	Date: -

Overview parts Tango XXL

Copyright reserved
by law


Unit of measure-
ment: mm

Lopital Nederland bv

Draw. no.: EXPV5850-2025-01

Armrest set width adjustment XL/XXL

Part no.	Art.no.	QTY.	Name	Drawing no.
1	70590011	2	Adjustment bar armrest	5900-19.2
2	70590006	1	Frame armrest XL XXL set	5900-16.2LR
3	70590046	2	Round bar armrest	5900-37.1
4	70590047	2	Unlock button armrest	5900-38
5	70820013	2	Cap	8200-23.1
6	70570503	2	Armrest PU	Froli armsteun - 5457-50 32 mm
7		2	Washer M25 DIN433 PA	M25 DIN 433 PA
8	70590048	2	Spring 0,8x11x80 Inox	Veer 0,8x11x80
9		2	Tube cap Ø28	Insteedkop rond 28 grijs
10		2	Circlip 25 DIN471	25 DIN 471
11		2	Screw M6x10 DIN914 A2	M6x10 DIN 914
12		2	Screw M5x10 DIN912 A2	M5x10 DIN912
13		2	Screw M6x16 ISO7380 A2	M6x16 ISO7380
14		4	Screw M6x35 DIN7991 A2	M6x35 DIN7991
15		2	Cylindric pin Ø8x36 DIN7	8x36 DIN 7 m6
16		2	Cylindric pin Ø8x30 DIN7	8x30 DIN 7

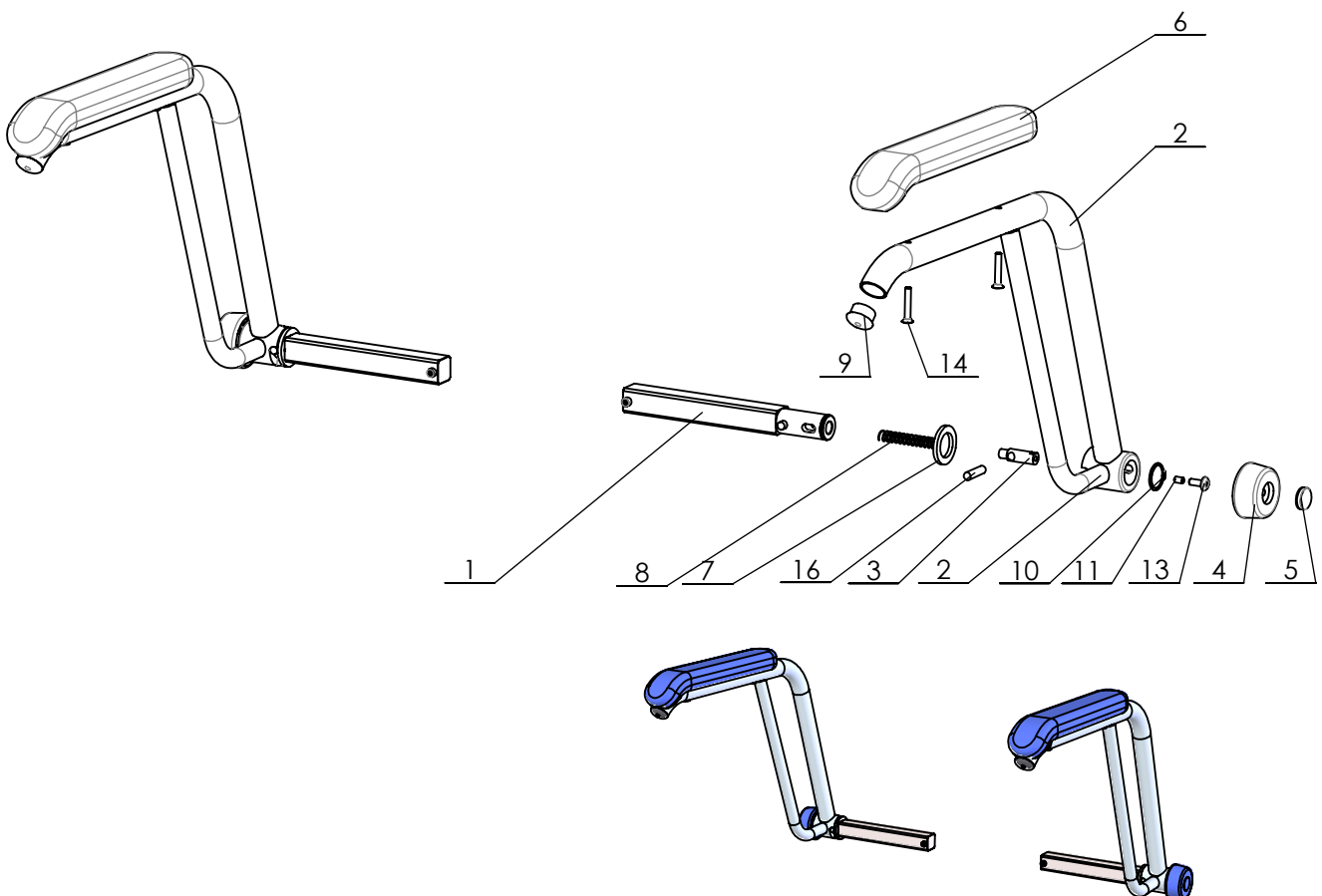

P:\SW-Fek\00 Stoei onderdelen\Armlegger breedteverstelling XL XXL

Drawn by:	KA	Date	27-11-2025
Revised	- /KA	Date	-
Revised	- /KA	Date	-

Overview parts Armrest set width adjustment XL/XXL




Copyright reserved by law Unit of measurement: mm

Lopital Nederland bv

Draw no.: EXPVArmleggerBVXLXXL-2025-01

Footrest set width adjustment XL/XXL

Part no.	Art.no.	QTY.	Name	Drawing no.
1	70590028	2	Frame width adjustment footrest	5900-18
2	70590027	2	Frame footrest XXL	5900-17
3	70590040	2	Frame footplate XXL	5900-40
4	70530032	1	Footplate set	5300-37LR.2
5	70530029	2	Pull knob M12x1.5	Treknop M12x1.5 - Kort
6		2	Tube cap 45x25	Insteekdop 25x25
7		2	Screw M5x10 DIN912 A2	M5x10 DIN912

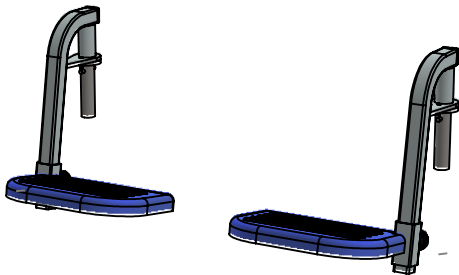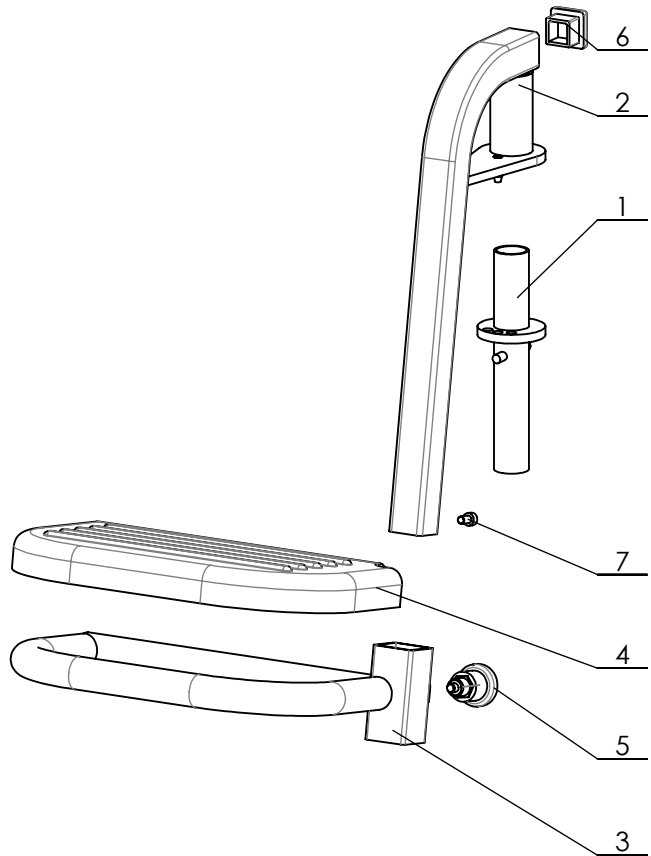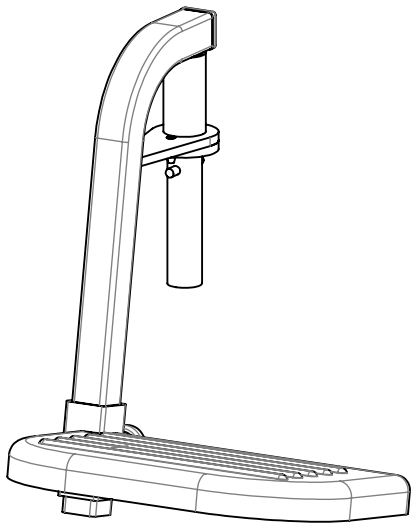

P:\SW-Fek\00 Stoei onderdelen\Voetsteun XXL

Drawn by:	KA	Date	27-11-2025
Revised	- /KA	Date	-
Revised	- /KA	Date	-

Overview parts Footrest set width adjustment XL/XXL

Copyright reserved
by law

Unit of measure:
mm

Lopital Nederland bv

Draw. no.: EXPVVoetsteunsetXXL-2025-01