

P:\SW-Fek\Vienna\Vienna 5035


Signed	KA	Date	07-10-2024
Revised	1 /KA	Date	11-11-2025
Revised	- /KA	Date	-

# Overview parts Vienna basic wall

Copyright reserved by law      Unit of measurement: mm

Lopital Nederland bv

Draw.no.:EXPV5035-2025-01

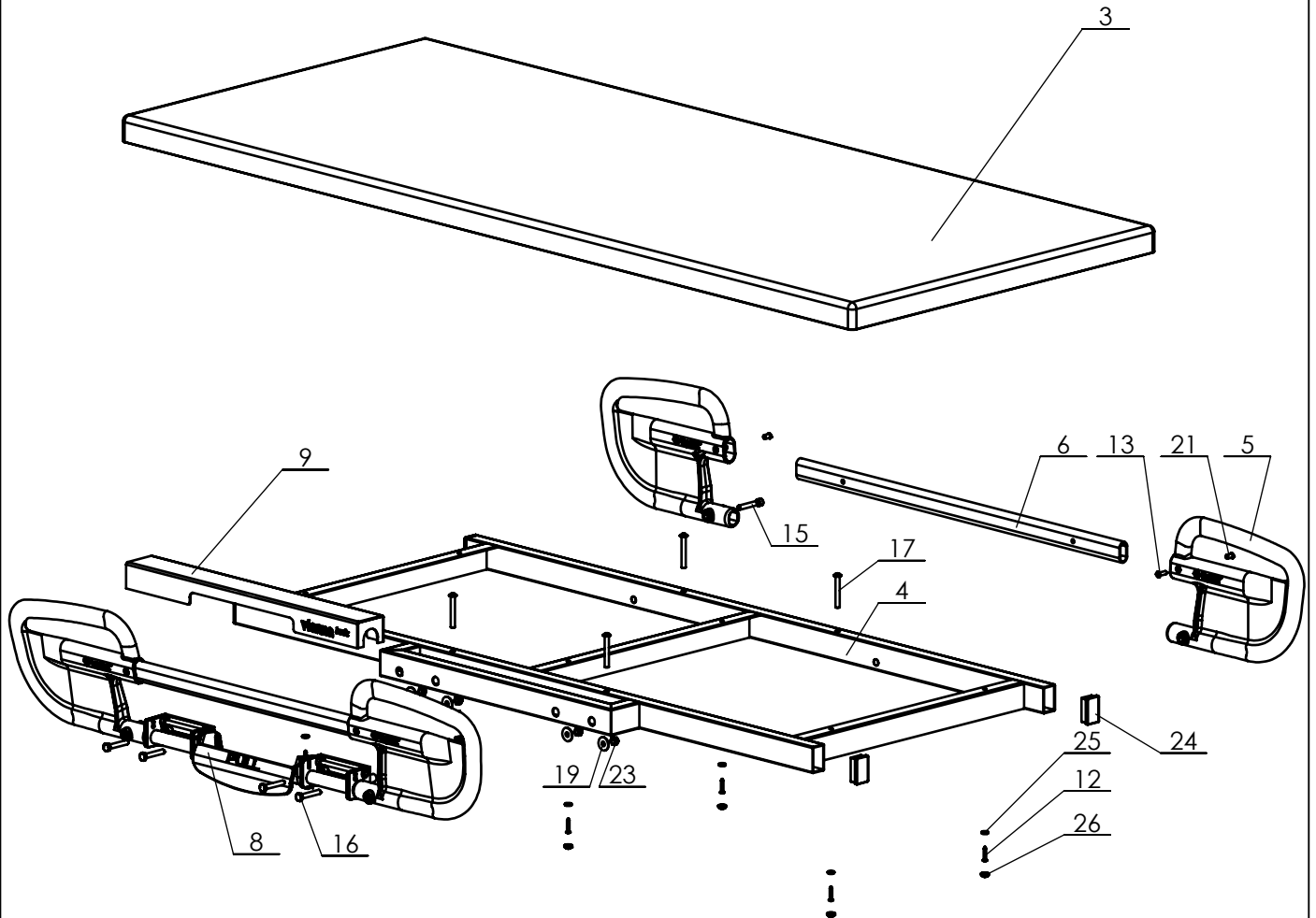

P:\SW-Tek\Vienna\Vienna 5035

Signed	KA	Date	07-10-2024
Revised	1 /KA	Date	11-11-2025
Revised	- /KA	Date	-
Copyright reserved by law		Unit of measurement: mm	


## Overview parts Vienna basic wall

Lopital Nederland bv

Draw.no.:EXPV5035-2025-01

## Vienna Basic wall-mounted fixed height dressing table , Article no: 6500 5035

Part no.	Art.no.	QTY.	Name	Drawing no.
1	70250004	2	Support arm	2500-12
2	70250010	2	Wall frame	2505-11
3	70503002	1	Mattress dressingtable 190x79	5035-12
4	70503501	1	Top frame	5035-11
5	70200026	2	Handle side rail/push bar	2000-35.3
6	70200019	1	Oval tube side rail	2000-28.2
7	70250018	2	Gas spring + ball joint radial	150mm 700N
8	67902020	1	Side rail compleet	2000-ZH
9	70503002	1	Cover side rail	5030-13
10	70250008	4	Bearing bush brass Ø16x11,2x8 mm	2500-16
11		1	Sticker Vienna basic	Sticker - Vienna basic
12		6	Screw 4,5x25 DIN 7981 A2	ST4,5x25 DIN7981
13		2	Screw M6x20 DIN 966 A2	M6x20 DIN965
14		2	Screw M6x65 DIN931 A2	M6x65 DIN931
15		2	Screw M8x50 DIN912 A2	M8x50 DIN912
16		4	Bolt M8x50 DIN933 A2	M8x50 DIN933
17		4	Screw M8x60 ISO7380 A2	M8x60 ISO7380
18		4	Washer M6 Ø18 DIN9021 A2	Carrosseriering M6 DIN9021
19		4	Washer M8 Ø25 A2	Carrosseriering M8
20		4	Washer M8 DIN125 PA	M8 DIN 125 PA
21		2	Sleeve nut M6x10x15 A2	M6 x10 x15
22		2	Locknut M6 DIN985 A2	M6 DIN985
23		6	Locknut M8 DIN985 A2	M8 DIN985
24		4	Tube cap 50x30	Insteekdop 50x30
25		6	Capring M5	Kapring M5
26		6	Screw cap M5	Afdekkap M5
27		4	Nut cap M6	Moerkap M6
28		2	Nutcap M8	Moerkap M8

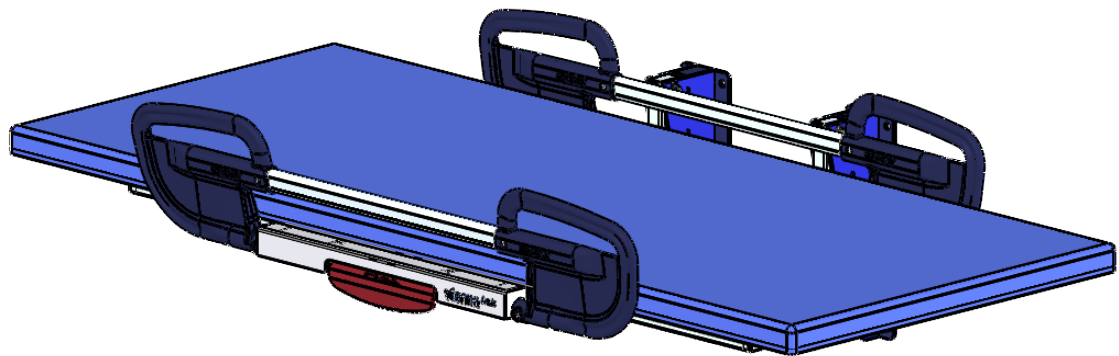

P:\SW-Tek\Vienna\Vienna 5035

Signed	KA	Date	07-10-2024
Revised	1 /KA	Date	11-11-2025
Revised	- /KA	Date	-

Copyright reserved  
by law

Unit of measure-  
ment: mm

## Overview parts Vienna basic wall

Lopital Nederland bv

Draw.no.:EXPV5035-2025-01