

Tango XL shower toilet chair

Instruction card

01. Safety recommendations

- Read the instruction for use before use (www.lopital.com).
- Maximum load 250 kg / 551.2 lbs.
- Make sure that the chair is on the brake during client transfers from and to the device.
- Place both feet on the foot supports.



02. Use

1. Place the chair in the correct position and turn the footrests away.
2. Apply the brakes to the chair!
3. If necessary, fold the armrests up.
4. Support the person with disability when they are taking a seat.
5. Shut the armrests and let the client's feet rest on the footplates.

Stand up by carrying out the same procedure in reverse order.

03. Removable seat

If required, the seat of the Tango XL can be removed from the frame. This allows the seat to be cleaned thoroughly, as no dirt can accumulate between the frame and the seat.




5100 5750

Tango XL




04. Wheel blocking

1. Wheel blocking
2. Unlocking

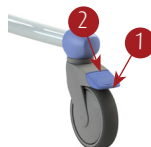

05. Directional- / braked wheel

1. To block the wheel direction (1x); to block the wheel (2x)
2. Unlocking

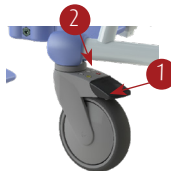


06. Footrest

Both foot plates must be folded away for the client to take a seat in the chair. Make sure that the client has both feet on the foot plates when moving.



07. Armrest

1. Release
2. Press the blue button
3. Turn backwards

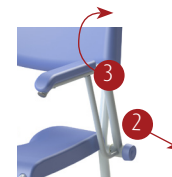

08. Toilet function

- *In combination with toilet:* the Tango XL can be moved over almost every toilet.
- *With bedpan:* the Tango XL can also be used as a toilet chair. Simply slide the stainless steel / plastic bedpan under the chair using the guides.

09. Cleaning and maintenance

The Tango XL must be cleaned after every use, with water and regular household cleaning agents. Do not use any abrasive or other aggressive cleaning products. If necessary, the PUR components can be disinfected with a solution of 70% ethanol or propanol. The complete maintenance schedule and cleaning protocol can be found in the instructions for use (www.lopital.com).

According to the regulation MDR (EU) 2017/745 for medical devices, Tango XL must be maintained in line with the manufacturer's instructions. Those instructions can be found in the maintenance protocol, see website Lopital (www.lopital.com).

Important!

This quick instruction card may not be considered a replacement for the instruction for use!

In the event of a breakdown, contact Lopital.

Tel: +31 (0) 13-5239300

E-mail: info@lopital.nl

Instruction for use
For the full instruction for use go to: www.lopital.com



ISO CERTIFIED
9001 - 13485 - 14001

INS5750-2026-01 EN